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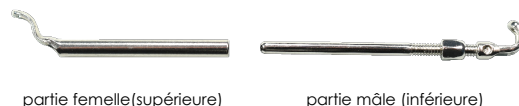
Dispositif médical

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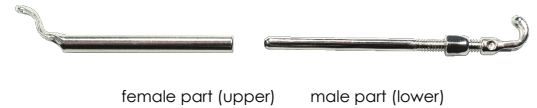
Date de fabrication

Reference and description of items concerned:

SPTR PUL 1 telescopic propulsion system (pair)

female part (upper) male part (lower)

Thank you for purchasing our PUL Twin system. **Please read these instructions carefully.** The materials used in the manufacture of the system have a perfect biological tolerance, validated within the framework of regulations relating to medical devices. Sold clean but not sterile, it is recommended that the cleaning rules in force in dental laboratories and surgeries be applied before use. Please observe the storage instructions.

Technical information :

Used to make a removable "PUL Twin" orthodontic appliance consisting of two resin trays. A pair of systems (male + female) is fixed to the framework tubes using support ball rods.

Clinical indication :

Orthodontic treatment of Class II malocclusions in growing children.

Please note:

Use is limited to orthodontic treatment. Use in patients is the responsibility of the practitioner.

Contraindication:

Not suitable for people allergic to Nickel and Cobalt

Clinical benefit :

Obtaining Class I

Laboratory

ASSEMBLY : We recommend that you read the manufacturing protocol from PUL Concept.

The assembly consists of connecting the lower splint to the upper splint (previously manufactured according to the PUL Concept protocol) with the PUL Twin system (ref SPTR) and attachments (ref TBS0.9) on the articulator in the corrected position (according to the practitioner's specifications).

- Place the clip in the part of the system designed for this purpose and insert the whole assembly into the gutter reinforcement tube, respecting the distance allowing the system to hinge freely (1mm), then fold the clip over and cut off the excess.
- In this way, attach the female part to the upper gutter and the male part to the lower gutter.

Practitioners

ACTIVATION : It is recommended that you have attended clinical training on the PUL concept with Dr CALLABE.

The PUL Twin system is activated by unscrewing the nut fitted on the lower male thread using the activation key (ref CABT); 3 turns correspond to 1mm of activation.

Patients

DEVICE PORT :

The device should be worn for 12 to 14 hours a day. The device should not be used for eating, brushing your teeth, playing wind music (flute, trumpet, etc.) or for sport, where there is a risk of breakage or loss.

To activate the retractors, follow the practitioner's instructions

CLEANING :

Before and after each use, clean the appliance under running water with a toothbrush (different from a toothbrush) to prevent tartar build-up.

PRECAUTIONS :

When you remove the appliance, always place it in its box to protect it. Never put it in a handkerchief or paper towel, as it may be thrown in the bin. For canteen meals, remember to remove the device, place it in its box and put it in the school bag so that it doesn't get lost. In the event of a cheek injury, slip a rolled compress between the cheek and the metal system. Do not place the device within the reach of a dog, as it could bite and break it.

Any serious incident involving a medical device from the Pul Concept® range must be reported immediately to your healthcare professional and to PUL Concept on +33 2 54 75 54 80 Email: pulconcept@pulconcept.com; and to the competent healthcare authority in your country.

 <p>PUL Concept ZA Les Plantes 7 rue Pierre et Marie Curie 41140 NOYERS SUR CHER - France 02.54.75.54.80 pulconcept@pulconcept.com</p>		 Medical device  Storage protected from the sun  Store in a dry place	 Contains Nickel and Cobalt  Reusable for one patient only  Manufacturing date
--	---	--	---

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