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Dispositif médical

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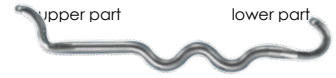
Date de fabrication

Reference and description of items concerned:

- SPWC PUL W stabilisation propulsion system - SHORT (pair)
- SPWL PUL W stabilisation propulsion system - LONG (pair)

Thank you for purchasing our PUL W system. **Please read these instructions carefully.** The materials used in the manufacture of the system have a perfect biological tolerance, validated within the framework of regulations relating to medical devices. Sold clean but not sterile, it is recommended that the cleaning rules in force in dental laboratories and surgeries be applied before use. Please observe the storage instructions.

Technical information :

Used to make a "PUL W removable anti-recidivism retainer" orthodontic appliance consisting of two resin splints. A pair of systems (identical connecting rods) is fixed to the framework tubes using support ball rods.

Please note:

Use is limited to orthodontic treatment. Use in patients is the responsibility of the practitioner.

Indication and clinical benefit :

Anti-recurrence stabilisation following orthodontic treatment.

Contraindication:

Not suitable for people allergic to Nickel and Cobalt

Laboratory

ASSEMBLY : We recommend that you read the manufacturing protocol from PUL Concept.

The assembly consists of connecting the lower splint to the upper splint (previously manufactured according to the PUL Concept protocol) with the PUL W system (ref SPWC/SPWL) and attachments (ref TBS0.9) on the articulator in the corrected position (according to the practitioner's specifications).

- Place the fastener in the part of the system provided for this purpose and insert the whole assembly into the gutter reinforcement tube, respecting the distance allowing the system to hinge freely (1mm), then fold the fastener over and cut off the excess.
- In this way, attach the upper part of the system to the upper gutter and the lower part of the system to the lower gutter.

Practitioners

ACTIVATION : It is recommended that you have attended clinical training on the PUL concept with Dr CALLABE.

The PUL W system is activated using orthodontic pliers.

3-nose pliers to close the W to shorten the system.

Flat nose pliers to open the W and lengthen the system.

Patients

DEVICE PORT :

The device must be worn every night for 12 months, then every other night for 6 months, then every third night for 6 months, then one night a week for a minimum of 12 months.

CLEANING :

Scrub the appliance under water with a toothbrush (different from the one used for teeth) to prevent tartar build-up.

PRECAUTIONS :

When you remove the appliance, always place it in its box to protect it. Never put it in a handkerchief or paper towel, as it may be thrown in the bin. In the event of a cheek injury, slip a rolled compress between the cheek and the metal system. Do not place the device within reach of a dog, as it could bite and break it.

Any serious incident involving a medical device from the Pul Concept® range must be reported immediately to your healthcare professional and to PUL Concept on +33 2 54 75 54 80 Email: pulconcept@pulconcept.com; and to the competent healthcare authority in your country.

 <p>PUL Concept ZA Les Plantes 7 rue Pierre et Marie Curie 41140 NOYERS SUR CHER - France 02.54.75.54.80 pulconcept@pulconcept.com</p>		<p> Medical device</p> <p> Storage protected from the sun</p> <p> Store in a dry place</p>	<p> Contains Nickel and Cobalt</p> <p> Reusable for one patient only</p> <p> Manufacturing date</p>
---	---	---	--

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