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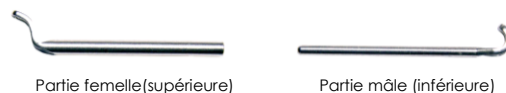
Dispositif médical

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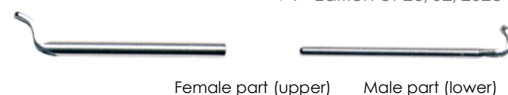
Date de fabrication

Reference and description of items concerned:

SP1T PUL 1 telescopic propulsion system (pair)

Thank you for purchasing our PUL 1 system. **Please read these instructions carefully.** The materials used in the manufacture of the system have a perfect biological tolerance, validated within the framework of regulations relating to medical devices. Sold clean but not sterile, it is recommended that the cleaning rules in force in dental laboratories and surgeries are applied before any use in patients. Please observe the storage instructions.

Intended use: intended for the orthodontic treatment of malocclusions in children and adults

Technical information :

Used to make a removable "PUL 1" orthodontic appliance consisting of two resin trays. A pair of systems (male + female) is fixed to the framework tubes using support ball rods.

Clinical benefit :

Obtaining Class I

Clinical indication :

Orthodontic treatment of Class II malocclusions in growing children.

Warning

Use is limited to orthodontic treatment. Use in patients is the responsibility of the practitioner.

Contraindication:

Not suitable for people allergic to Nickel and Cobalt

Laboratory

ASSEMBLY : We recommend that you read the manufacturing protocol from PUL Concept.

The assembly consists of connecting the lower splint to the upper splint (previously manufactured according to the PUL Concept protocol) with the PUL 1 system (ref SP1T) and attachments (ref TBS0.9) on an articulator in the corrected position (according to the practitioner's specifications).

- Place the clip in the part of the system designed for this purpose and insert the whole assembly into the gutter reinforcement tube, respecting the distance allowing the system to hinge freely (1mm), then fold the clip over and cut off the excess.
- In this way, attach the female part to the upper gutter and the male part to the lower gutter.

Practitioners

ACTIVATION : It is recommended to have attended the clinical training on the PUL concept with Dr CALLABE.

2 options to choose from:

- The PUL 1 system is activated by adding a propulsion spring (ref RPCC) to the inside of the female part of the PUL 1 system using a spring pusher (ref PR). The spring can be removed using the spring puller (ref TR).

- Propulsion shims (ref CPF1.5 for 1.5 mm and CPF2.0 for 2 mm) are added to the male part using a shim positioner (ref PCP).

RECOMMENDATIONS FOR USE :

For moderate cases (< 3mm): wear at night. For severe cases (inter-maxillary misalignment > 3 mm): night-time wear and, if possible, a few hours during the day, but no more than 14 hours a day.

Patients

DEVICE PORT :

The device must be worn in accordance with the practitioner's recommendations.

The PUL can be uncomfortable for the first few days when it comes to speaking and swallowing saliva, but after that adaptation is rapid. You need to remove the device to eat, brush your teeth, play wind music (flute, trumpet, etc.) and for sport where there is a risk of breakage or loss. It's a mobile device, so it's normal for it not to sit well on your teeth, and for it to come off if you open your mouth wide.

CLEANING :

Before and after each use, clean the appliance under running water with a toothbrush (different from a toothbrush) to prevent tartar build-up.

PRECAUTIONS :

When you remove the appliance, always place it in its box to protect it. Never put it in a handkerchief or paper towel, as it may be thrown in the bin. For canteen meals, remember to remove the device, place it in its box and put it in the school bag so that it doesn't get lost. In the event of a cheek injury, slip a rolled compress between the cheek and the metal system. Do not put the device within reach of a dog, as it could bite and break it.

Any serious incident involving a medical device from the Pul Concept® range must be reported immediately to your healthcare professional and to PUL Concept on +33 2 54 75 54 80 Email: pulconcept@pulconcept.com; and to the competent healthcare authority in your country.

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