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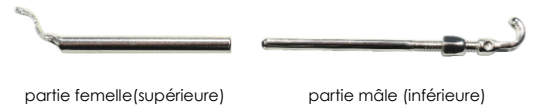
Dispositif médical

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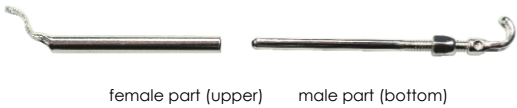
Date de fabrication

Reference and description of items concerned:

SPT PUL 1 telescopic propulsion system (pair)

Thank you for purchasing our PUL Twin system. **Please read these instructions carefully.** The materials used in the manufacture of the system have a perfect biological tolerance, validated in accordance with medical device regulations. Sold clean but not sterile, we recommend that you apply the cleaning rules in force in dental laboratories and practices before use. Please observe storage instructions.

Intended use: for orthodontic treatment of malocclusions in children and adults

Technical information :

Used to make a removable "PUL Twin" orthodontic appliance consisting of two resin splints. A pair of systems (male + female) is attached to the framework tubes using support ball rods.

Clinical benefit :

Class I certification

Clinical indication :

Orthodontic treatment of Class II malocclusions in growing children.

Warning:

Use is limited to orthodontic treatment. Patient use is the responsibility of the practitioner.

Contraindication:

Not suitable for people allergic to Nickel and Cobalt

Laboratory

ASSEMBLY : We recommend that you read the manufacturing protocol from PUL Concept.

The assembly consists of connecting the lower splint to the upper splint (previously manufactured according to the PUL Concept protocol) with the PUL Twin system (ref SPTR) and attachments (ref TBS0.9) on an articulator in corrected position (according to the practitioner's specifications).

- Place the clamp in the appropriate part of the system and insert the whole assembly into the gutter's reinforcement tube, maintaining the distance required for free articulation of the system (1mm), then fold the clamp over and cut off the excess.
- In this way, attach the female part to the upper gutter and the male part to the lower gutter.

Practitioners

ACTIVATION : It is recommended to have attended a clinical training session on the PUL concept with Dr CALLABE.

The PUL Twin system activated by unscrewing the nut mounted on the lower male thread using the activation key (ref CABT); 3 turns correspond to 1mm of activation.

Patients

DEVICE PORT :

The device should be worn for 12 to 14 hours a day. Do not use the device for eating, brushing your teeth, playing wind music (flute, trumpet, etc.), or for sports, where there is a risk of breakage or loss.

For retractor activation, follow the 's instructions

CLEANING :

Before and after each use, clean the appliance under running water with a toothbrush (different from a toothbrush) to prevent tartar build-up.

PRECAUTIONS :

When removing the device, always place it in its box to protect it. Never put it in a handkerchief or paper towel, as it may be thrown in the garbage can. For canteen meals, remember to remove the device, place it in its box and put it in the school bag to avoid losing it. In the event of cheek injury, slide a rolled compress between the cheek and the metal system. Do not place the device within reach of a dog, as it could bite and break it.

Any serious incident involving a Pul Concept® medical device must be reported immediately to your healthcare professional and to PUL Concept at +33 2 54 75 54 80 Email: pulconcept@pulconcept.com ; and to the competent healthcare authority in your country

 PUL Concept ZA Les Plantes 7 rue Pierre et Marie Curie 41140 NOYERS SUR CHER - France 02.54.75.54.80 pulconcept@pulconcept.com	 0297	 MD Medical device  Storage protected from sunlight  Store in a dry place  Contains Nickel and Cobalt  Reusable for one patient only  Date of manufacture
---	---	---

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